

SYNCHRON® System(s)
Chemistry Information Sheet

ALP
Alkaline Phosphatase
REF (200 tests/cartridge) 442670
REF (400 tests/cartridge) 476821

For In Vitro Diagnostic Use

ANNUAL REVIEW

Reviewed by:	Date	Reviewed by:	Date

PRINCIPLE

INTENDED USE

ALP reagent, when used in conjunction with SYNCHRON LX® System(s), UniCel® DxC 600/800 System(s), is intended for the quantitative determination of alkaline phosphatase activity in human serum or plasma.

CLINICAL SIGNIFICANCE


Alkaline phosphatase measurements are used in the diagnosis and treatment of liver, bone, parathyroid, and intestinal diseases.

METHODOLOGY

ALP reagent is used to measure alkaline phosphatase activity by a kinetic rate method using a 2-amino-2-methyl-1-propanol (AMP) buffer.^{1,2,3,4,5,6} In the reaction, alkaline phosphatase catalyzes the hydrolysis of the colorless organic phosphate ester substrate, p-nitrophenylphosphate, to the yellow colored product, p-nitrophenol, and phosphate. This reaction occurs at an alkaline pH of 10.3.

The SYNCHRON® System(s) automatically proportions the appropriate sample and reagent volumes into the cuvette. The ratio used is one part sample to 50 parts reagent. The system monitors the change in absorbance at 410 nanometers. This change in absorbance is directly proportional to the activity of ALP in the sample and is used by the System to calculate and express ALP activity.

CHEMICAL REACTION SCHEME

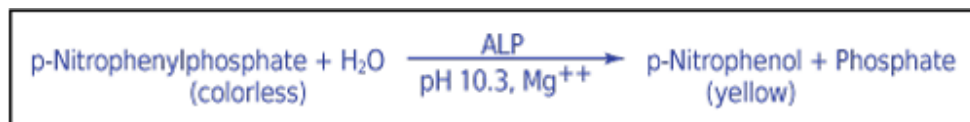

E015176L.EPS

SPECIMEN

TYPE OF SPECIMEN

Biological fluid samples should be collected in the same manner routinely used for any laboratory test.⁷ Freshly drawn serum or plasma are the specimens of choice. Acceptable anticoagulants are listed in PROCEDURAL NOTES section of this chemistry information sheet. Whole blood is not recommended for use as a sample.

SPECIMEN STORAGE AND STABILITY

1. Tubes of blood are to be kept closed at all times and in a vertical position. It is recommended that the serum or plasma be physically separated from contact with cells within two hours from the time of collection.⁸
2. Separated serum or plasma should not remain at room temperature longer than 8 hours. If assays are not completed within 8 hours, serum or plasma should be stored at +2°C to +8°C. If assays are not completed within 48 hours, or the separated sample is to be stored beyond 48 hours, samples should be frozen at -15°C to -20°C. Frozen samples should be thawed only once. Analyte deterioration may occur in samples that are repeatedly frozen and thawed.⁸

ADDITIONAL SPECIMEN STORAGE AND STABILITY CONDITIONS AS DESIGNATED BY THIS LABORATORY:

--

SAMPLE VOLUME

The optimum volume, when using a 0.5 mL sample cup, is 0.3 mL of sample. For optimum primary sample tube volumes and minimum volumes, refer to the Primary Tube Sample Template for your system.

CRITERIA FOR UNACCEPTABLE SPECIMENS

Refer to the PROCEDURAL NOTES section of this chemistry information sheet for information on unacceptable specimens.

CRITERIA FOR SAMPLE REJECTION AS DESIGNATED BY THIS LABORATORY:

--

PATIENT PREPARATION

SPECIAL INSTRUCTIONS FOR PATIENT PREPARATION AS DESIGNATED BY THIS LABORATORY:

SPECIMEN HANDLING

SPECIAL INSTRUCTIONS FOR SPECIMEN HANDLING AS DESIGNATED BY THIS LABORATORY:

REAGENTS

CONTENTS

Each kit contains the following items:

Two ALP Reagent Cartridges (2 x 200 tests) or (2 x 400 tests)

VOLUMES PER TEST

Sample Volume	5 µL
ORDAC Sample Volume	3 µL
Total Reagent Volume	250 µL
Cartridge Volumes	
A	228 µL
B	22 µL
C	--

REACTIVE INGREDIENTS

REAGENT CONSTITUENTS

p-Nitrophenylphosphate	15 mmol/L
2-Amino-2-methyl-1-propanol	350 mmol/L

Also non-reactive chemicals necessary for optimal system performance.

Avoid skin contact with reagent. Use water to wash reagent from skin.

MATERIALS NEEDED BUT NOT SUPPLIED WITH REAGENT KIT

At least two levels of control material

Saline

REAGENT PREPARATION

No preparation is required.

ACCEPTABLE REAGENT PERFORMANCE

The acceptability of a reagent is determined by ensuring that quality control results are within your facility's acceptance criteria.

REAGENT STORAGE AND STABILITY

ALP reagent when stored unopened at +2°C to +8°C, will obtain the shelf-life indicated on the cartridge label. Once opened, the reagent is stable for 10 days at +2°C to +8°C unless the expiration date is exceeded. DO NOT FREEZE.

REAGENT STORAGE LOCATION:

--

CALIBRATION

CALIBRATOR REQUIRED

Calibration is not required.

TRACEABILITY

This measurand (analyte) is traceable to the manufacturer's selected Measurement Procedure as described in the Methodology section.

QUALITY CONTROL

At least two levels of control material, normal and abnormal, should be analyzed daily. In addition, these controls should be run with each new reagent cartridge and after specific maintenance or troubleshooting procedures as detailed in the appropriate system manual. More frequent use of controls or the use of additional controls is left to the discretion of the user based on good laboratory practices or laboratory accreditation requirements and applicable laws.

The following controls should be prepared and used in accordance with the package inserts. Discrepant quality control results should be evaluated by your facility.

TABLE 1 QUALITY CONTROL MATERIAL

CONTROL NAME	SAMPLE TYPE	STORAGE

CONTROL NAME	SAMPLE TYPE	STORAGE

TESTING PROCEDURE(S)

1. If necessary, load the reagent onto the system.
2. Program samples and controls for analysis.
3. After loading samples and controls onto the system, follow the protocols for system operations.

For detailed testing procedures, refer to the SYNCHRON LX *Operations Manual*, or the UniCel DxC 600/800 System *Instructions For Use* (IFU) manual.

CALCULATIONS

The SYNCHRON® System(s) performs all calculations internally to produce the final reported result. The system will calculate the final result for sample dilutions made by the operator when the dilution factor is entered into the system during sample programming.

REPORTING RESULTS

Equivalency between the SYNCHRON LX and UniCel DxC 600/800 Systems has been established. Chemistry results between these systems are in agreement and data from representative systems may be shown.

REFERENCE INTERVALS

Each laboratory should establish its own reference intervals based upon its patient population. The following reference intervals were taken from literature and a study performed on SYNCHRON Systems.^{9,10}

TABLE 2 REFERENCE INTERVALS

INTERVALS	SAMPLE TYPE	CONVENTIONAL UNITS	S.I. UNITS
Literature	Serum or Plasma	38 – 126 IU/L	0.65 – 2.14 µkat/L
SYNCHRON	Serum or Plasma	32 – 91 IU/L	0.55 – 1.55 µkat/L

INTERVALS	SAMPLE TYPE	CONVENTIONAL UNITS	S.I. UNITS
Laboratory			

Refer to References (11,12,13) for guidelines on establishing laboratory-specific reference intervals.

--

ANTICOAGULANT TEST RESULTS

Table 3 Acceptable Anticoagulants^a

ANTICOAGULANT	LEVEL TESTED FOR IN VITRO INTERFERENCE	AVERAGE PLASMA-SERUM BIAS (IU/L)
Ammonium Heparin	29 Units/mL	NSI ^b
Lithium Heparin	29 Units/mL	NSI
Sodium Heparin	29 Units/mL	NSI

None identified.

1. The following substances were tested for interference with this methodology:

SUBSTANCE	SOURCE	LEVEL TESTED	OBSERVED EFFECT ^d
Bilirubin (unconjugated)	Bovine	30 mg/dL	NSI ^e
Hemoglobin	RBC hemolysate	300 mg/dL	-7 @ 34 IU/L
		400 mg/dL	-13 @ 170 IU/L
		500 mg/dL	NSI @ 901 IU/L
Lipemia	Intralipid ^f	500 mg/dL	NSI

2. Inhibitors of alkaline phosphatase activity include: oxalates, Hg^{++} , excess inorganic phosphate, bile acids, some amino acids (e.g., phenylalanine), and urea.
3. Refer to References (14,15,16) for other interferences caused by drugs, disease and preanalytical variables.

Analytic Range

The SYNCHRON® System(s) method for the determination of alkaline phosphatase provides the following analytical range:

TABLE 5 ANALYTICAL RANGE

SAMPLE TYPE	CONVENTIONAL UNITS	S.I. UNITS
Serum or Plasma	5 – 1000 IU/L	0.1 – 16.7 μ kat/L
Serum or Plasma (ORDAC) ^g	800 – 1650 IU/L	13.3 – 27.5 μ kat/L

Samples with activities exceeding the high end of the analytical range should be rerun with ORDAC enabled or diluted with saline and reanalyzed.

REPORTABLE RANGE (as determined on site):

TABLE 6 REPORTABLE RANGE

SAMPLE TYPE	CONVENTIONAL UNITS	S.I. UNITS

SENSITIVITY

Sensitivity is defined as the lowest measurable concentration which can be distinguished from zero with 95% confidence. Sensitivity for ALP determination is 5 IU/L (0.08 μ kat/L).

EQUIVALENCY

Equivalency was assessed by Deming regression analysis of patient samples to accepted clinical methods.

Serum or Plasma (in the range of 9 to 971 IU/L):

Y (SYNCHRON LX Systems)	= 1.015X + 2.96
N	= 79
MEAN (SYNCHRON LX Systems)	= 178.8
MEAN (SYNCHRON CX7 DELTA)	= 173.2
Correlation Coefficient (r)	= 0.9990

Refer to References (17) for guidelines on performing equivalency testing.

PRECISION

A properly operating system should exhibit precision values less than or equal to the following:

TABLE 7 PRECISION VALUES

TYPE OF PRECISION	SAMPLE TYPE	1 SD		CHANGEOVER VALUE ^h		% CV
		IU/L	μ kat/L	IU/L	μ kat/L	
Within-run	Serum/Plasma	3.0	0.05	85.7	1.43	3.5
	Serum/Plasma (ORDAC)	NA ⁱ	NA	NA	NA	10.0
Total	Serum/Plasma	4.5	0.08	85.7	1.43	5.3
	Serum/Plasma (ORDAC)	NA	NA	NA	NA	15.0

Comparative performance data for a SYNCHRON LX[®] System evaluated using the NCCLS Proposed Guideline EP5-T2 appears in the table below.¹⁸ Each laboratory should characterize their own instrument performance for comparison purposes.

TABLE 8 NCCLS EP5-T2 PRECISION ESTIMATE METHOD

TYPE OF IMPRECISION	SAMPLE TYPE		No. Systems	No. Data Points ¹	Test Mean Value (IU/L)	EP5-T2 Calculated Point Estimates	
						SD	%CV
Within-run	Serum	Control 1	1	80	36.5	1.36	3.71
	Serum	Control 2	1	80	136.5	1.73	1.27
	Serum	Control 3	1	80	239.3	2.24	0.94
Total	Serum	Control 1	1	80	36.5	1.68	4.61
	Serum	Control 2	1	80	136.5	2.65	1.94
	Serum	Control 3	1	80	239.3	4.05	1.69

NOTICE

These degrees of precision and equivalency were obtained in typical testing procedures on a SYNCHRON LX[®] System and are not intended to represent the performance specifications for this reagent.

ADDITIONAL INFORMATION

For more detailed information on SYNCHRON LX Systems or UniCel DxC Systems, refer to the appropriate system manual.

SHIPPING DAMAGE

If damaged product is received, notify your Beckman Coulter Clinical Support Center.

REFERENCES

1. Kay, H. D., *J. Biol. Chem.*, 89:235 (1930).
2. King, E. J., Armstrong, A. R., *Can. Med. Assoc. J.*, 31:376 (1934).
3. Kind, P. R. N., King, E. J., *J. Clin. Path.*, 7:322 (1947).
4. Bessey, O. A., Lowry, O. H., Brock, M. J., *J. Biol. Chem.*, 164:321 (1947).
5. Babson, A. L., et al., *Clin. Chem.*, 12:482 (1966).
6. Bowers, G. N., McComb, R. B., *Clin. Chem.*, 12:70 (1966).
7. Tietz, N. W., "Specimen Collection and Processing; Sources of Biological Variation", *Textbook of Clinical Chemistry*, 2nd Edition, W. B. Saunders, Philadelphia, PA (1994).
8. National Committee for Clinical Laboratory Standards, *Procedures for the Handling and Processing of Blood Specimens*, Approved Guideline, NCCLS publication H18-A, Villanova, PA (1990).
9. Tietz, N. W., *Clinical Guide to Laboratory Tests*, 3rd Edition, W. B. Saunders, Philadelphia, PA (1995).
10. "Working Group on Enzymes", *Eur. J. Clin. Chem. Clin. Biochem.*, 30:247 256 (1992).
11. National Committee for Clinical Laboratory Standards, *How to Define, Determine, and Utilize Reference Intervals in the Clinical Laboratory*, Approved Guideline, NCCLS publication C28-A, Villanova, PA (1995).
12. Tietz, N. W., ed., *Fundamentals of Clinical Chemistry*, 3rd Edition, W. B. Saunders, Philadelphia, PA (1987).
13. Henry, J. B., *Clinical Diagnosis and Management by Laboratory Methods*, 18th Edition, W. B. Saunders Company, Philadelphia, PA (1991).
14. Young, D. S., *Effects of Drugs on Clinical Laboratory Tests*, 4th Edition, AACC Press, Washington, D. C. (1995).
15. Friedman, R. B., Young, D. S., *Effects of Disease on Clinical Laboratory Tests*, 3rd Edition, AACC Press, Washington, D.C. (1997).
16. Young, D. S., *Effects of Preanalytical Variables on Clinical Laboratory Tests*, 2nd Edition, AACC Press, Washington, D. C. (1997).
17. National Committee for Clinical Laboratory Standards, *Method Comparison and Bias Estimation Using Patient Samples*, Approved Guideline, NCCLS publication EP9-A, Villanova, PA (1995).
18. National Committee for Clinical Laboratory Standards, *Precision Performance of Clinical Chemistry Devices*, Tentative Guideline, 2nd Edition, NCCLS publication EP5-T2, Villanova, PA (1992).



Beckman Coulter Ireland Inc., Mervue Business Park, Mervue, Galway, Ireland (353 91 774068)



Beckman Coulter, Inc., 250 South Kraemer Blvd., Brea, CA 92821

ENDNOTES

- a Data shown was collected using SYNCHRON CX Systems. Equivalency between SYNCHRON LX Systems has been established by Deming regression analysis to SYNCHRON CX Systems.
- b NSI = No Significant Interference (within ± 6.0 IU/L or 7%).
- c Data shown was collected using SYNCHRON CX Systems. Equivalency between SYNCHRON LX Systems has been established by Deming regression analysis to SYNCHRON CX Systems.
- d Plus (+) or minus (-) signs in this column signify positive or negative interference.
- e NSI = No Significant Interference (within ± 6 IU/L or 7%).
- f Intralipid is a registered trademark of KabiVitrum, Inc., Clayton, NC 27250.
- g Overage Detection and Correction. Refer to the SYNCHRON LX Operations Manual, or the UniCel DxC 600/800 System Instructions For Use (IFU) manual for more details on this function.
- h When the mean of the test precision data is less than or equal to the changeover value, compare the test SD to the SD guideline given above to determine the acceptability of the precision testing. When the mean of the test precision data is greater than the changeover value, compare the test % CV to the guideline given above to determine acceptability. Changeover value = (SD guideline/CV guideline) x 100.
- i NA = Not applicable.
- j The point estimate is based on the pooled data from one system, run for twenty days, two runs per day, two observations per run on an instrument operated and maintained according to the manufacturer's instructions.